


USE OF Bonopty® Adapter 1:3 Revolutions


1 Start procedure as per normal routine.

a Press penetration cannula with stylet down to bone.

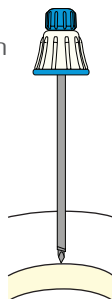


b Replace stylet with eccentric drill.

Rotate a few turns until drill tip catches the bone.
Verify position and direction using diagnostic imaging.

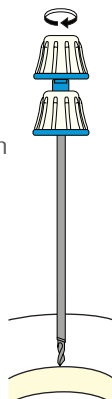

c Retract penetration cannula to expose drill tip beyond penetration cannula to allow the eccentric movement of the drill.

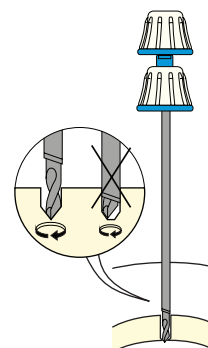


2 Attach Bonopty Adapter 1:3 Revolutions to the drill hub.

Grip the base of the adapter using finger and thumb grip.

Turn the top of the adapter and feel the eccentric drill turning. **Use moderate pressure only.**

After drill has passed through cortical bone, anchor penetration cannula 2-3 mm into cortex.

Remove drill and adapter.

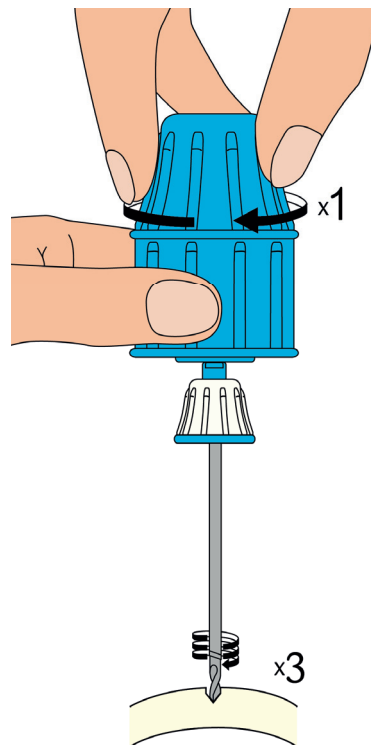

Using Bonopty® Adapter 1:3 Revolutions, one complete turn of the top of the handle...

...corresponds to three turns of the drill tip

3 Insert biopsy needle through the anchored cannula and continue procedure as per normal routine.

Before using Bonopty® Adapter 1:3 Revolutions read the instructions for use which accompany the product for indications, contraindications, warnings and precautions. Bonopty® is a registered trademark of AprioMed AB.

29255 Rev 01 2014-04

HEAD OFFICE
AprioMed AB
Virdings Allé 28
SE-754 50 Uppsala, Sweden
Phone: +46 18 430 14 40
Free Fax (Int'l): +46 770 339972

E-mail: info@apriomed.com
www.apriomed.com

AprioMed

US SALES OFFICE
AprioMed, Inc.
2 Palmer Dr, Suite 1
Londonderry, NH 03053, USA
Phone: +1 603 421 0875
Fax (US toll free): 866 239 1797