


SURG-N-FLOW

REF **SNF-100**

Intended use / Utilisation prévue:


SURG-N-FLOW handles and sets are primarily designed for orthopaedic surgery and combine suctioning function with filtering, removing bone fragments, chips, cement, marrow, and blood clots from the operative area. It must be attached to a jar/canister by a tube before connecting to a vacuum source. This product is single use only.

Les poignées et ensembles SURG-N-FLOW sont principalement conçus pour la chirurgie orthopédique et combinent la fonction d'aspiration avec filtrage, élimination des fragments d'os, des copeaux, du ciment, de la moelle et des caillots sanguins de la zone opératoire. Ce doit être attaché à un bocal/bidon par un tube avant de se connecter à une source de vide. Ce produit est à usage unique.

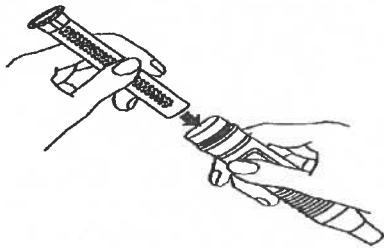


To replace top, unscrew, remove old filter, insert new as indicated by arrows.

Pour remplacer le haut, dévisser, retirer l'ancien filtre, insérer le nouveau comme indiqué par des flèches.

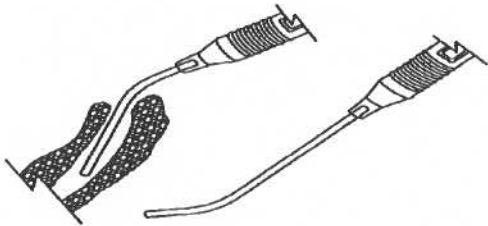


Two (2) suction tips are included that may be inserted into the handle tip in four (4) different configurations. After insertion, twist slightly to seal tip.

Deux (2) embouts d'aspiration sont inclus qui peuvent être inséré dans le bout du manche en quatre (4) configurations différentes. Après l'insertion, tournez légèrement pour sceller la pointe.

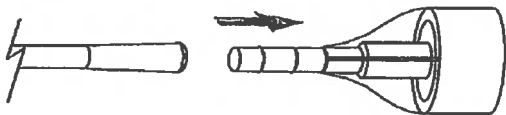

Fit flexible tubing to top end (not included).

Monter le tube flexible sur l'extrémité supérieure (non inclus).

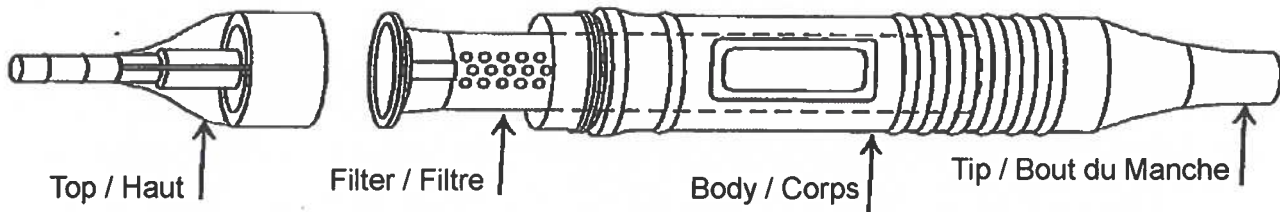

Manufactured and distributed by:
Fabriqué et distribué par :



INSTRUMED
SURGICAL

2180 Dunwin Drive, Units 5 & 6
Mississauga, ON L5L 5M8 Canada



Consult
Instructions
For Use



Do Not Use if
Package is Opened
or Damaged



Single Use



Do Not
Restenize



Not made with
latex rubber

STERILE | R

Rx only

